PELO F30 Duo

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Vlastnosti

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Producent:

WENZHOU LONGWAN MEDICAL DEVICE FACTORY
Unit 4, Building 27, No. 706 Yanyun Road
Oujiangkou Industrial Cluster District, 325026, Wenzhou, Zhejiang P.R.C
info@taiyumed.com

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Shanghai International Holding Corp. Sp. z o.o.
Eiffestrasse 80, 20537 Hamburg, Niemcy

Importer:

CM Trade Via s.r.o.
Olomoucká 3419/7, 618 00 Brno, Republika Czeska
tel.: +420 549 274 164, info@cm-trade.cz

PELO_IB_F30 Duo_ver.00
Wersja: 20/03/2025

EN

PELO F30 Duo

DUAL HEAD STETHOSCOPE

Features

- Ergonomic design
- High quality
- Good acoustic response
- Non-cooling membrane
- PVC eartips
- Double-sided membrane head for high and low frequency sound detection
- Aluminum alloy lyre with chrome-plated band
- Large membrane diameter: 46 mm
- Small membrane diameter: 42.8 mm

Used to listen to various sounds of the heart, lungs and organs in the human body. Class I medical device
Lightweight aluminum alloy head. Adjustable chrome-plated aluminum alloy lyre. Durable one-piece molded „Y“ PVC hose.

Safety Notice

A stethoscope is not a substitute for professional medical diagnosis. If you have any concerns about your health, always consult a doctor. Keep the stethoscope out of reach of children. If used incorrectly or damaged, the stethoscope may lead to incorrect results and misinterpretation of body sounds.

Cleaning, maintenance and storage:

Clean the stethoscope thoroughly after use to prevent the transfer of bacteria or infections. To clean the head, tubing and ear tips, we recommend using a clean cloth or cotton swab moistened with diluted alcohol or isopropyl alcohol for effective removal of dirt. Protect the stethoscope from high humidity, direct sunlight and physical shock. Do not store or use the stethoscope near heaters or open flames.

Disposal:

When disposing of the stethoscope, follow the regulations currently in force in your region. The manufacturer does not specify any specific conditions for disposal.

Liability for Defects:

This stethoscope is covered by a liability for defects for a period of 24 months from the date of purchase. Liability for defects does not apply to goods damaged due to improper use or repairs carried out by an unqualified person.

Transport and storage conditions: 10–40 °C | 15–85 % relative humidity

Manufacturer:

WENZHOU LONGWAN MEDICAL DEVICE FACTORY
Unit 4, Building 27, No. 706 Yanyun Road
Oujiangkou Industrial Cluster District, 325026, Wenzhou, Zhejiang P.R.C
info@taiyumed.com

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Shanghai International Holding Corp. GmbH