# stylies®

## Comfort & Care

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Safety instructions bandage: **Important!**

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The green dot – label marking the adhesion to a system contribution to the recycling of waste

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Complies with European Directives on safety and EMC

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If you suffer from a medical problem (e.g. varicose veins, thrombosis, etc.), consult a doctor prior to using the product.

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Never apply to helpless, infants or heat-sensitive individuals.

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Inspect that all components of the product are in a faultless condition prior to each start-up of the product. The individual components of this product may not be disassembled, repaired or modified.

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Do not insert any needles or pointy objects into the product.

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The product is not intended for professional, medical or hospital use.

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Persons with reduced physical, sensory or mental capabilities, lack of experience or knowledge may only use the appliance if they are supervised by a person who is responsible for their safety or if they have been shown how to use the appliance safely and they have understood the risks inherent in its use.

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Children should not play with the product. Keep packaging (e.g. plastic bags) away from children.

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Always switch off the appliance: whenever it is left unattended, before cleaning, when it fails to work properly, and after use.

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For safety reasons, do not use accessories that are not recommended or sold by the manufacturer.

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If the product is used for purposes other than those specified by the manufacturer, operated incorrectly, or not repaired professionally, we will not accept any liability for any resulting damage. In this case, the warranty claim will be void.

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Never use the product with open wounds, burns, hematoma, swellings, or similar conditions.

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Stop immediately, if you notice any unpleasant sensations, redness of the skin, rashes or inflammation. Heatsensitive persons in particular should use the product with care and interrupt the treatment in a timely manner.

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For sensitive skin, it is strongly recommended not to use directly on the skin. Constantly check the condition of the skin to avoid any redness or burns.

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Use the product only in dry conditions and with a dry body – never in the bathtub, shower, above filled sinks, in pools, saunas, or during rain.

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Do not fall asleep during use. Don't use overnight.

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Don't use the product while folded or wrinkled. Avoid over-folding the bandage. Do not put heavy objects on the bandage.

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The product should not be used in the following cases:

- Health or disability issues
- Insensitivity to heat
- Injury, sunburn, scarring, inflamed areas or other damage due to disease
- Phlebitis, arthritis or other inflammatory processes
- Under the effects of alcohol, sleeping pills, sedatives or drugs
- Pacemaker

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Use only a 5V power adapter for charging, otherwise the device may get damaged.

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Children should not play with the appliance. Keep the appliance and power cable away from children under 8 years. Keep packaging (e.g. plastic bags) away from children.

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If the appliance is used for purposes other than those specified by the manufacturer, operated incorrectly, or not repaired professionally, we will not accept any liability for any resulting damage. In this case, the warranty claim will be void.

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Don't directly connect any contacts as this can lead to short circuits.

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Never use close to water. Do not expose to rain or humidity. Never immerse the appliance into water or other liquids.

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Check the appliance and power cable for damage. Never switch on a damaged appliance (including power cable) – have it repaired or replaced by the manufacturer, the manufacturer's service centre or by a qualified service technician. Never open the appliance yourself – risk of injury!

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Never place the appliance or the power cable on a hot surface (hob). Keep the appliance away from open flames. Do not expose the appliance to intense heat (heat sources, radiators, sunshine).

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Load the appliance only at a socket with correct voltage. Never use extension leads.

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For the first time use or when the Powerbank has been out of use for a long time, about 5–6 full charges and discharges are needed to achieve full performance.

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When charging, make sure that the Powerbank is sufficiently ventilated.

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This appliance is not intended for commercial use.

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The Powerbank should not be short-circuited or brought into contact with metal.

 Rozsah provozních teplot: -10 °C–50 °C.  
Operating temperature range: -10 °C–50 °C.

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 Nevkládejte do úst.  
Don't put in the mouth.

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Disposal: Not domestic waste! Appliance must be disposed of in compliance with local regulations. Make disused appliances immediately unusable. Unplug and cut through the power cable. Take electrical appliances to the point of sale or to a collection point.

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Warranty:

The product guarantee corresponds with the respective local law and regulations and applies for at least 2 years from date of purchase. It covers faults in construction, workmanship and material. Excluded from this guarantee are all wear parts as well as inappropriate use or maintenance as well as modifications or repairs done by unauthorised third parties.

Please keep your proof of purchase to verify any claims under guaranty.

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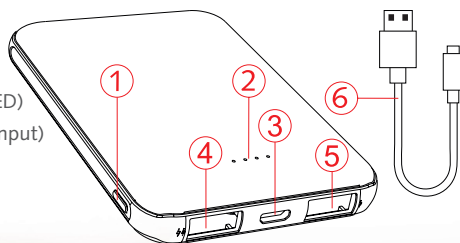
Connect the USB cable (⑤) which is in the bandage pocket (①) to the Powerbank (④ or ⑤).

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Put on the bandage and secure it with the velcro strips (⑥). Please ensure that the heating bandage sits loosely, i.e. some air circulates between the heating bandage and your skin/clothes during use. Accumulation of heat must be prevented.

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Switch on by pressing the power switch (⑧). The device starts at the lowest temperature (green light / 35 °C).

To change the temperature, press the power switch (⑧) again as follows:

- 1 x for medium temperature (blue light / 48 °C)
- 2 x for high temperature (red light / 60 °C)

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To switch off, press the power switch (⑧) till the light turns off.

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We recommend using the bandage for 30 to 45 minutes on the highest heat or 2 hours on the lowest heat and this no more than 2–3 times per day. Due to the automatic shutdown, the heating bandage turns off automatically after 2 hours.

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On the inside of the bandage is a net pocket for the use of a cold pack (not included). Follow the original instructions of the cool pack manufacturer for use.

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**Important:** Avoid over-folding the bandage. Do not put heavy objects on the bandage.

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For simple cleaning of the bandage, use a soft brush. The heating bandage can be washed in the washing machine. Use the fine washing program of your machine, which washes at a maximum of 30 °C. The washing machine should only be lightly loaded, as the washing program requires a lot of water for cleaning. Please observe the washing instructions below.

**Warning:** please remove the Powerbank before washing! Before washing, slide the USB cable into the pocket for the Powerbank.

**Info:** the cable and the USB connection will not be damaged. The controllers are sealed so that no water gets inside when washing.

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Coplax AG